



### Implant Supported Full Arch Hybrid Restoration Schedule

**1<sup>st</sup> Appointment – Open Tray Implant Level Impression:** Provide lab with the analogs if you have them or we can Order them for you.

The lab needs 5-10 working days before next appointment

(multi-unit abutments may be needed to correct the angulation of the implant, to allow for proper draw and to make sure the screw holes come out on the occlusal or lingual. We will provide them for an additional fee if needed)

**2<sup>nd</sup> Appointment – Verification Jig:** Make sure the impression is accurate (should fit passively) you need to X-Ray to verify. If it doesn't fit, please cut, dura-lay and take a new open-tray impression with jig in place. Additional verification Jig appointments will be needed until it fits properly – lab will need 5 working days in the lab for each correction and try-in (Not needed for Conus Abutment Denture)

Bite Rim – we provide this screw retained bite rim to help determine the proper bite and achieve the proper placement of the teeth. Use the conventional denture technique to determine proper lip support, mark the midline, incisal length, where the central should be, and the corners of the mouth.

After Jig and Bite Rim are accepted the lab will need at least 5-10 working days before next appointment

**3<sup>rd</sup> Appointment – Try-In:** We will have either denture teeth or PMMA teeth for you to try-in. Evaluate the bite, shape, shade, location of the teeth, and amount of tissue display. Look at the midline and check phonetics. This may take multiple times to get everything correct – lab will need 5 working days in the lab for each correction and try-in

After final tooth set-up is accepted the lab will need at least 15-20 working days before next appointment

**4<sup>th</sup> Appointment – Framework:** We reverse engineered from the denture teeth and setup and fabrication of the Titanium or zirconium substructure and set the teeth on the substructure. Please Do another try-in to verify passive fit of the framework and double check the bite, Shape, shade, and location of teeth. (Not needed for Conus Abutment Denture)

Once accepted the lab will need at least 5-10 working days before the final appointment

**5<sup>th</sup> Appointment – Final Seat:** Delivery of the final Prosthesis - Please torque to the manufactures recommended Settings.

Please keep the model work for any further work needed on the case like re-treading.  
(denture's last approximately 7 years)